

Warnings & Cleaning Precautions:

Do not attempt to service this product yourself. Opening the housing may expose you to risk of shock, among other risks.

To avoid damaging the lens due to high temperature or humidity, do not store the Dino-Lite otoscope outdoors without adequate protection.

Use and store the Dino-Lite otoscope within this temperature range:
(32°F ~ 104°F) / (0°C ~ 40°C)

If water has entered the Dino-Lite otoscope, unplug it immediately.

To avoid tripping or entanglement, please be aware of cable placement and do not leave the cable dangling.

Disposable specula tips included. To repurchase, look for:
KleenSpec Disposable Specula:
5mm (WEL52135EA), 4mm (WEL52134EA), 3mm (WEL52133EA)

To avoid potential damage to the camera case, painted surfaces, and lens, do not use any cleaning detergent or volatile solution. Scan the QR code with your phone for cleaning instructions or visit:
<https://www.dinolite.us/en/how-do-i-clean-the-dino-lite>



Contact Us:
Phone: 888-668-2442
+1-310-618-9990
Email: support@dinolite.us
Website: www.dinolite.us

Medical Series

Otoscope HF4113-EUT

APPLICATIONS

| Point-of-care | Telemedicine | Self-monitoring | Prophylaxis

Dino-Lite
Digital Microscope

Otoscope HF4113-EUT

Attach disposable specula to the otoscope.



Gently place the device into the ear. Face the Dino-Lite logo upwards for proper image orientation.



Plug the USB into the computer and launch the software. Adjust the focus by rotating the dial.



For a wireless connection, the WF-20 Wi-Fi adapter is sold separately.

