

Warnings & Cleaning Precautions:

Do not attempt to service this product yourself. Opening the housing may expose you to risk of shock, among other risks.

To avoid damaging the lens due to high temperature or humidity, do not store the Dino-Lite capillarscope outdoors without adequate protection.

Use and store the Dino-Lite capillarscope within this temperature range: (32°F ~ 104°F) / (0°C ~ 40°C)

If water has entered the Dino-Lite capillarscope, unplug it immediately.

To avoid tripping or entanglement, please be aware of cable placement and do not leave the cable dangling.

Cap cleaning: To clean oil left on the cap, remove the cap from the device, and wash it with detergent solution or 75% alcohol.

Cleaning the body of the scope: To avoid potential damage to the camera case, painted surfaces, and lens, do not use any cleaning detergent or volatile solution. Scan the QR code with your phone for cleaning instructions or visit:
<https://www.dinolite.us/en/how-do-i-clean-the-dino-lite>



Contact Us:
Phone: 888-668-2442
+1-310-618-9990
Email: support@dinolite.us
Website: www.dinolite.us

Medical Series

Capillarscope HF4115-N2UT(W)

APPLICATIONS

| Point-of-care | Telemedicine | Self-monitoring | Prophylaxis

Dino-Lite
Digital Microscope

Capillaryscope HF4115-N2UT(W)

Attach the curved cap to the capillaryscope.



Plug the USB into the computer and launch the software. Adjust the focus by rotating the dial.



Gently place the device over the nail fold. Face the Dino-Lite logo towards the examinee for proper image orientation.



- Tips
- * Use oil, such as baby oil, to enhance image results.
 - * Rotate the polarizer to the '+' position to reduce glare.
 - * At touching distance, it will focus at around 50x and 200x.

For a wireless connection, the WF-20 Wi-Fi adapter is sold separately.

