

## Warnings & Cleaning Precautions:

Do not attempt to service this product yourself. Opening the housing may expose you to risk of shock, among other risks.

To avoid damaging the lens due to high temperature or humidity, do not store the Dino-Lite dermascope outdoors without adequate protection.

Use and store the Dino-Lite dermascope within this temperature range: (32°F ~ 104°F) / (0°C ~ 40°C)

If water has entered the Dino-Lite dermascope, unplug it immediately.

To avoid tripping or entanglement, please be aware of cable placement and do not leave the cable dangling.

For external use only.

Cleaning the body of the scope: To avoid potential damage to the camera case, painted surfaces, and lens, do not use any cleaning detergent or volatile solution. Scan the QR code with your phone for cleaning instructions or visit:

<https://www.dinolite.us/en/how-do-i-clean-the-dino-lite>



Contact Us:

Phone: 888-668-2442

+1-310-618-9990

Email: [support@dinolite.us](mailto:support@dinolite.us)

Website: [www.dinolite.us](http://www.dinolite.us)

# Medical Series

## Dermascope HF4115ZT(W)-D

### APPLICATIONS

| Point-of-care | Telemedicine | Self-monitoring | Prophylaxis

**Dino-Lite**  
Digital Microscope

## Dermascope HF41 15ZT(W)-D

Attach interchangeable cap to the dermascope.



Plug the USB into the computer and launch the software. Adjust the focus by rotating the dial.



Gently place the device onto the skin. Face the Dino-Lite logo away from the examiner for proper image orientation.



Tip\* To reveal follicles, rotate the polarizer to the '-' position. Alternatively, rotate it to the '+' position to view under the epidermis.

For a wireless connection, the WF-20 Wi-Fi adapter is sold separately.

